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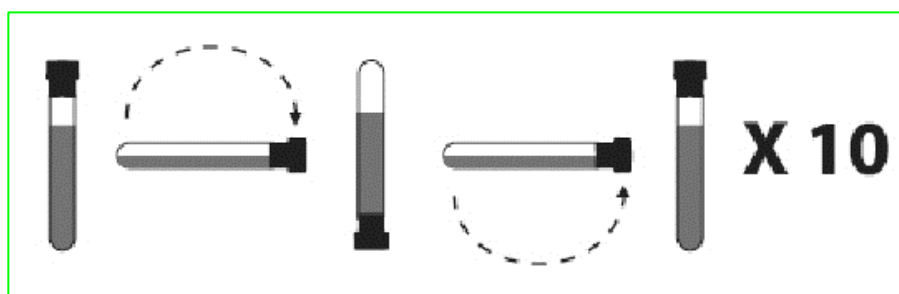
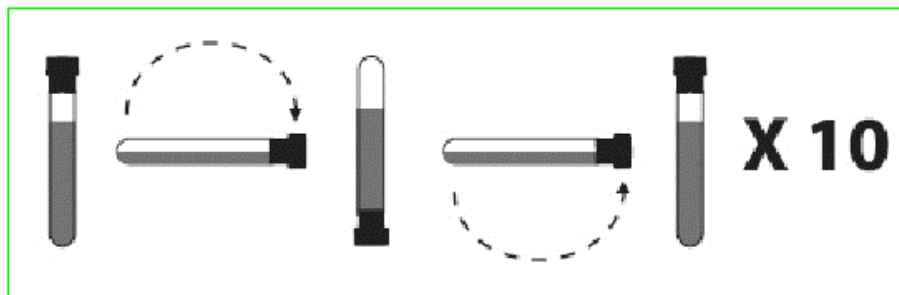
The laboratory is authorized according to the french legislation to perform cytogenetics (including molecular cytogenetics), genetics, biochemical (including maternal serum screening) and infectious diseases testing for fetus.

CONDITIONS FOR COLLECTING AND SENDING THE SAMPLE

1. Identify the 2 « Streck » tubes provided.
2. Collect 2 x 10 ml of whole blood. The 2 tubes must be **completely full**.
It is normal to have a **low flow**.
3. Gently rotate the tubes by at least 10 times.

4. Put the 2 tubes into the « **refrigerated** » plastic bag (transparent).
5. Store the plastic bag between **+ 4° C and + 8° C**.
The tubes will be shipped between + 4°C and + 8°C.
THE SAMPLE MUST BE RECEIVED AT LABORATOIRE CERBA WITHIN 4 DAYS.
6. **DO NOT FREEZE.**
7. Insert the following documents into the external pocket of the plastic bag:
 - **The test request form fulfilled by the prescriber**
 - **Information certificate signed by the prescriber and the informed consent signed by the patient**
 - **The ultrasonic report showing the gestational age**

“Streck” tubes lifetime before use:

- **Until the date mentioned onto the tube at room temperature**
- **Do not freeze the tubes**

Laboratoire Cerba is not liable for cancellation of the requested test in case of extension of transport time nor for non-respect of the minimum time of pregnancy required (in this case: invoiced).